SYNOVIUM HCS

PRODUKTINFORMATION - SV

BESKRIVNING:

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PRODUCT INFORMATION - EN

DESCRIPTION:

SYNOVIUM HCS is a sterile, apyrogenic and physiological solution of non-reticulated hyaluronic acid of non-animal origin (bacterial fermentation) and of chondroitin sulphate of animal origin (marine).

This solution is available in a single-use, pre-filled syringe.

Each box contains one SYNOVIUM HCS syringe, a sterile, single-use 21G (0.8x50 mm) needle (only for the intra-articular injection of the device) and a package leaflet.

COMPOSITION:

SYNOVIUM HCS	3 mL
Sodium hyaluronate	60 mg
Chondroitin sulphate	60 mg
Sodium chloride	21 mg
Borate buffer pH 7.2 q.s. ad (sodium borate, boric acid, water for injections)	q.s. 3 mL

STERILISATION:

SYNOVIUM HCS syringes are moist-heat sterilised in their individual sterility protector.

The 21G needles are sterilised with ethylene oxide.

INDICATIONS:

SYNOVIUM HCS is indicated in the symptomatic treatment of knee osteoarthritis.

PROPERTIES:

SYNOVIUM HCS is administered by intra-articular injection and its therapeutic effect can be ascribed to visco-supplementation, a process by which the rheological properties of the synovial fluid of the arthritic joint are improved. SYNOVIUM HCS reduces pain and restores joint mobility by replacing and supplementing the elasto-viscosity of the synovial fluid in the arthritic joint.

CONTRAINDICATIONS:

SYNOVIUM HCS must not be used in patients with:

- known hypersensitivity to one of the ingredients (in particular sodium hyaluronate and chondroitin sulphate),
- a pre-existing skin infection at the planned injection site.

All inflammatory joint conditions should be treated prior to starting treatment with intra-articular injection of SYNOVIUM HCS.

PRECAUTIONS FOR USE:

SYNOVIUM HCS is indicated for intra-articular injection only. Always aspirate a small amount of synovial fluid prior to injection to ensure the intra-articular location of the needle. Do not inject intra-vascularly.

Do not inject outside the joint cavity or into the synovial tissue or capsule or if there is a large amount of effusion. Do not inject any other products at the same time as SYNOVIUM HCS.

As with all injections, special caution is necessary in patients receiving ongoing anticoagulant treatment due to an increased risk of bruising.

As with any intra-articular injection, patients are advised to rest for 24 hours and avoid any form of activity, sports-related or otherwise.

SYNOVIUM HCS has not been tested in pregnant or breast-feeding women; it is therefore not recommended for use in these patients.

INCOMPATIBILITIES:

The solution contains hyaluronic acid (sodium hyaluronate), which is incompatible with quaternary ammonium salts such as benzalkonium chloride. SYNOVIUM HCS must never come into contact with this type of product or with surgical material that has been cleaned with this type of product.

UNDESIRABLE EFFECTS:

The practitioner must inform the patient about immediate or delayed potential adverse effects related to the injection of this device prior to the intervention.

The various adverse effects are as follows:

- Immediate adverse effects related to the injection: pain, redness, swelling at the injection site.
- Adverse effects with an inflammatory origin: pseudo-acute infectious arthritis, chondrocalcinosis, granulomatous synovitis.

The application of an ice pack on the joint can help reduce immediate adverse effects related to the injection (pain, redness, swelling at the injection site).

The onset and/or persistence of adverse effects must be reported by the patient to the practitioner as soon as possible, and the practitioner must administer a suitable treatment.

Any other adverse effect related to the injection of the device must be reported to the distributor and/or the manufacturer.

METHOD OF USE:

SYNOVIUM HCS is intended for intra-articular injection by an authorised and specialised practitioner in accordance with applicable local regulations.

The practitioner's technique is essential to treatment success. This device must be used by practitioners who have received specific training in injection techniques.

The device must be used in its original packaging as provided. Any change or use of this product outside of the conditions for use defined in this leaflet can jeopardise its sterility, and its efficacy and tolerance will not be guaranteed.

The patient must be informed by the practitioner of the indications, contraindications, incompatibilities and potential adverse effects of the device before undertaking any treatment. The injection must be given by a practitioner in accordance with the rules concerning asepsis.

Procedure for use:

- Before the injection, the area to be treated must be thoroughly disinfected.
- Verify that the sterility protector is intact.
- Take the syringe and remove the protective tip.
- Hold the body of the syringe and insert the needle in the Luer Lock system.
- Gently turn the needle in a clockwise direction in order to secure the needle in the Luer Lock system.
- The fixation of the needle must be checked visually.
- Remove the cap of the needle.
- Inject slowly.

Non-compliance with these instructions can result in the detachment of the needle and/or leaking of the device around the Luer Lock.

DOSAGE:

SYNOVIUM HCS is for adults only.

The dosage regimen of SYNOVIUM HCS is one intra-articular injection that can be repeated if the patient's signs and symptoms require it.

WARNINGS:

- Do not use if the product's individual sterility protector has been damaged.
 - Check the expiry date on the label.
 - Do not reuse, as sterility cannot be guaranteed.
 - Do not resterilise.
 - Use of the provided needle is recommended.
- The practitioner will be solely responsible for the use of any other needles.
- The needles and syringes used must be disposed of in suitable containers and the current regulations complied with for their removal.

STORAGE CONDITIONS:

- Store between +2°C and 25 °C.
- Do not expose to light or freezing temperatures.

SYNOVIUM HCS is a medical device CE 0459 - Manufactured in France by LCA S.A.

The 21G 0.8 mm x 50 mm needle is a medical device CE 0050 – Manufactured by Becton Dickinson or CE 0123 – Manufactured by Terumo.

Year of CE Mark authorisation: 2022. Date this product information leaflet was last revised: 04-2024. PN: 910-1061/03 .

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SYNOVIUM HCS ist ein Medizinprodukt (CE 0459) – Hergestellt in Frankreich von LCA S.A.